


BJ-241

Participation box

User guide

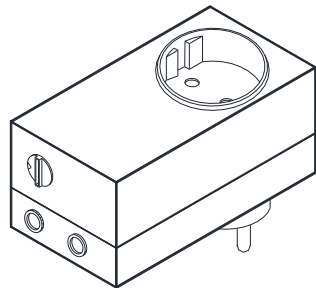

www.bjliveat.com

1. Description of the device

This device provides switch control of any electrical appliance.

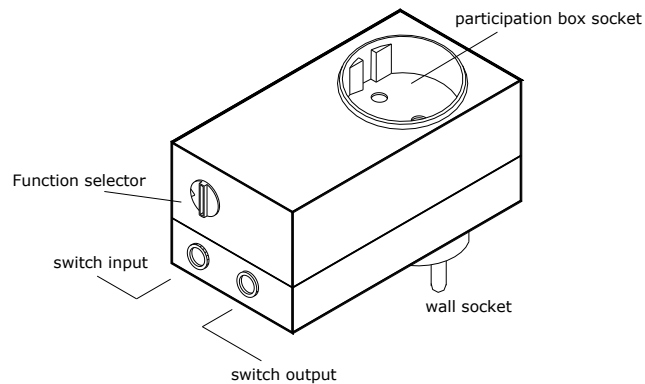


Fig.1: Participation box

2. Technical data

Power Supply	230~ / 50Hz
Maximum power supply	250V~
Dimensions	105x59x50 mm
2 x switch 3.5 mm mono jack plug	
Maximum load:	
- Resistive loads: 2000W/9A ($\cos\phi=1$)	
- Motor loads: 1000W/5,5A ($\cos\phi=0,8$)	
- Halogen lamps with transformer: 300W/1,7A ($\cos\phi=0,76$)	
Maximum switch output charge: 24V - 1A	

3. Operating instructions

To use the Participation Box you must follow the next steps:

1. Plug the participation box into the wall socket, having in mind the technical specifications of the equipment.

2. Connect the device you have chosen to participation box socket (fig.1), having in mind the technical specifications of the equipment and being sure it is in operating condition.
3. Situate the function selector (fig.1) in the desired position according to the desired action (table 1).
4. Connect a switch to the "switch input" (fig.1) and activate it.
5. If you want, now you can control a switch adapted device while you activate the appliance connected to the Participation Box.

The switch output will work at the same time and in the same way as the appliance connected to the Participation Box. Connect the device to the "switch output" (fig.1) with the included cable and activate it.

Table 1

Function selector position	Operating mode
0	If the Participation Box was on it goes off and vice versa
1	During the click the Participation Box is on
2	Timed 1s
3	Timed 2s
4	Timed 5s
5	Timed 10s
6	Timed 15s
7	Timed 30s
8	Timed 45s
9	Timed 60s
A	Timed 1min 30s
B	Timed 2min
C	Timed 2min 30s
D	Timed 3min
E	Timed 4min
F	Timed 5min

4. General information

Maintenance

The Enabler can be cleaned using a damp soft cloth. Always carry out this operation with the Enabler disconnected.

In the event of a fault in the Enabler, do not handle it and contact your distributor.

Disposal

Dispose of the waste product via a collection point for electronic scrap or via your specialist dealer. Any unauthorized tampering with, or modifications to the product will annul this warranty.



Warranty

Within the statutory warranty period we undertake to rectify free of charge by repair or replacement any product defects arising from material or production faults.

Any unauthorized tampering with, or modifications to the product will annul this warranty.

Conformity

This product complies with the essential requirements of the Directives 2004/108/EC and 2006/95/CEE.

The Declaration of Conformity can be downloaded from the product's webpage at www.bjliveat.com.

Customer Service

If, despite of correct handling, faults or malfunctions occur or if the product was damaged, please contact us at the address below:

BJ Adaptaciones

C/ Mare de Déu del Coll, 70, Bajos

08023 Barcelona

España

Phone: +34 93 285 04 37

Fax: + 34 93 553 56 34

e-Mail: info@bjliveat.com

www.bjliveat.com

Dealer information

The logo for BJ Live! features a stylized blue 'b' inside a circle to the left of the letters 'BJ' in a large, bold, blue font. Below 'BJ' is the word 'Live!' in a similar blue font, with an exclamation mark. At the bottom, the tagline 'smart solutions that suit you' is written in a smaller, blue, sans-serif font.

BJ
Live!
smart solutions
that suit you